Mölnlycke®

# Mepilex® Border Post-Op

WITH SAFETAC® TECHNOLOGY

Single use  
Usage unique

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# Mepilex® Border Post-Op

Self-adherent soft silicone surgical dressing

## Product description

Mepilex Border Post-Op is a self-adherent dressing that absorbs blood exudate and minimises the risk for maceration. The waterproof outer layer protects the wound from dirt and bacteria. The dressing has a Safetac® wound contact layer that is a unique adhesive technology. It minimises pain to patients and trauma to wounds and the surrounding skin at dressing removal.

Mepilex Border Post-Op consists of:

- a transparent outer film which is breathable but waterproof, providing a barrier to external contaminants
- a flexible absorbent pad in two layers: a non-woven spreading layer and a layer with super absorbent fibres
- a wound contact layer consisting of soft silicone adhesive (Safetac) and a film carrier

## Dressing material content:

Silicone, polyurethane, viscose, polyester, polyolefins, polyacrylate and superabsorbent acrylate fibers.

## Indications for use

Mepilex Border Post-Op is a self-adhesive absorbent surgical dressing designed for exuding wounds. It is intended for acute wounds, such as surgical wounds, cuts and abrasions.

## Precautions

- If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.
- Always consult a health care professional before using Mepilex Border Post-Op on Epidermolysis Bullosa patients.
- Do not use Mepilex Border Post-Op on patient and/or user with known hypersensitivity to the ingoing materials/components of the product.

- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package (sterile barrier) is damaged or opened prior to use. Do not re-sterilise.

## Instructions for use

Mepilex Border Post-Op can be used by lay persons under supervision of health care professionals.

1. Cleanse the wound and the surrounding skin in accordance with clinical practice.
2. Dry the surrounding skin thoroughly.
3. The Mepilex Border Post-Op wound pad should overlap the wound by at least 1-2 cm in order to protect the surrounding skin from maceration and fixate the dressing securely.
4. For dressings with three release films, remove the middle release film first. Apply the adherent side to the wound, allowing initial fixation of the dressing. For dressings with two release films, grip both overlapping release films, exposing adhesive surface allowing initial fixation of the dressing.
5. Avoid wrinkles while gently removing the remaining release films, start with the longer part. Do not stretch the dressing during application.

Mepilex Border Post-Op may be left in place for up to 7 days depending on the condition of the wound and the surrounding skin, or as indicated by clinical practice.

Disposal should be handled according to local environmental procedures.

## Other information

If any serious incident has occurred in relation to Mepilex Border Post-Op, it should be reported to Mölnlycke Health Care.

Mepilex® and Safetac® are registered trademarks of Mölnlycke Health Care AB.

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